DE—FILTRATIONS-GESICHTSMASKE

ANWEISUNGEN

Executive Standard: EN149:2001+A1:2009

Filterklasse: FFP2 NR

Verwendung:

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Fabricante:

Baoding Yinhong Yuhe Medical Device Manufacturing Co., Ltd
Nanlongshan Village, Dawangdian Industrial Park, Xushui District, Baoding City, Hebei Province, China.

Importador:

Europapa Handels GmbH
Am Bahndamm 5, 41334 Nettetal, Alemania.

EN—FILTERING HALF MASK

INSTRUCTIONS

Executive standard: EN149:2001+A1:2009

Filter class: FFP2 NR

Usage:

1. Separate the edges of the mask to fully open it.
2. Slightly bend the nose wire to form a gentle curve.
3. Hold the mask upside down to expose the two ear loops.
4. Cup the mask under your chin, and pull the ear loops over your ears.
5. Conform the nose piece across the bridge of your nose by firmly pressing down with your fingers.
6. Continue to adjust the mask and secure the edges until you feel you have achieved a good facial fit.

Warnings:

1. Single use only, do not reuse.
2. Before use, the wearer must learn the Directions for Application in accordance with applicable safety and health standards.
2. If the mask becomes damaged or breathing becomes difficult, leave the area. Discard and replace the mask.
3. For safety, keep away from the source of fire.
4. Do not touch the inner body of the mask.

Storage condition:

Do not store in highly humid or damp areas. It is recommended that the storage temperature be -30—+40°C, and the humidity be no more than 50%.

Shelf life:

Filtering Half Mask is valid for three years under the above storage condition.

Manufacturer:

Baoding Yinhong Yuhe Medical Device Manufacturing Co., Ltd
Nanlongshan Village, Dawangdian Industrial Park, Xushui District, Baoding City, Hebei Province, China.

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Fabricant:

Baoding Yinhong Yuhe Medical Device Manufacturing Co., Ltd
Nanlongshan Village, Dawangdian Industrial Park, Xushui District, Baoding City, Hebei Province, Chine.

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Produttore:

Baoding Yinhong Yuhe Medical Device Manufacturing Co., Ltd
Nanlongshan Village, Dawangdian Industrial Park, Xushui District, Baoding City, Hebei Province, Cina.

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